

Home Use




Bubble Humidifier Instructions for Use and Cleaning Instructions

7100 Bubble Humidifier- 3 PSI Pop-off | 7600 Bubble Humidifier- 6 PSI Pop-off

The Bubble Humidifier is a disposable, single-patient use device for use in health care facilities and home care. The humidifier is used to provide additional humidity to supplemental oxygen therapy and is available with either a 3 PSI or 6 PSI safety pop-off valve.

Instructions for Use

1. Wash hands and remove bubble humidifier from package.
2. Unscrew lid and add water to the Maximum Level marking on the side of the bottle. Sterile, distilled or boiled water is recommended.
3. Replace lid and ensure lid is screwed on tight.
4. Attach bubble humidifier to oxygen source, i.e. flowmeter, concentrator, liquid tank.
5. Test for leaks (A). Turn the flow to 6 LPM and block the outlet port on the bubble humidifier. An alarm should sound in 5 seconds or less. If not, check connections.
6. Attach oxygen tubing to the outlet port (B) and adjust flow to prescribed setting.
7. Apply interface, i.e. nasal cannula or oxygen mask, to patient.
8. If water level drops to Minimum level marking on bottle, add more water.

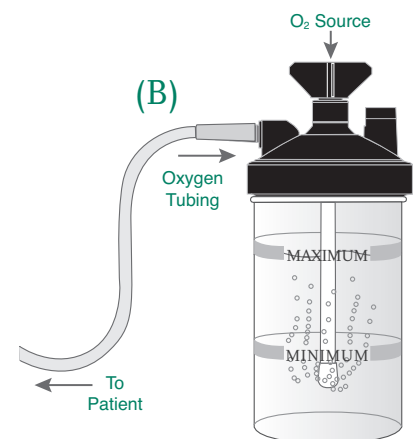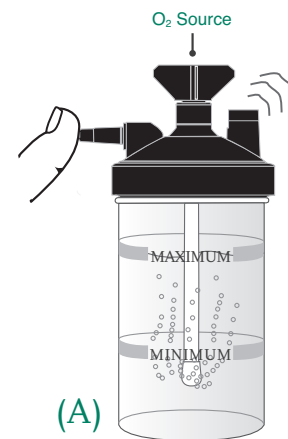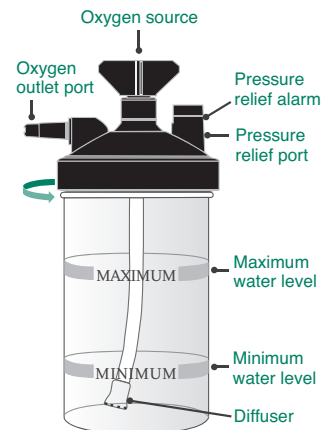


Bubble Humidifier Instructions for Use and Cleaning Instructions (continued)

Cleaning Instructions


9. Every 14 days, wash your bubble humidifier with warm soapy water.
10. Rinse thoroughly with distilled or boiled water.
11. Recommend replacing Bubble Humidifier every 30 days.

Note Tap Water Use: If using Tap Water in your bubble humidifier or rinsing with Tap Water after cleaning, wash the bubble humidifier every 3 days.

Final Setup

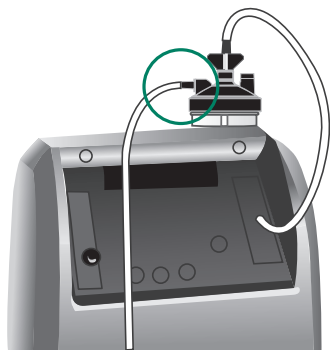

Your setup may vary slightly depending on what system you use for your oxygen source.

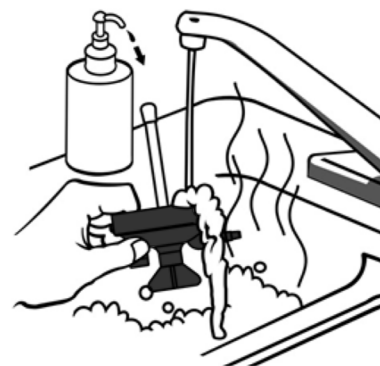

“The Bubble Humidifier is a disposable, single-patient use device for use in health care facilities and home care.”